
Disposable Port Closure Device

CL401, CL402, CL403, CL404, CL405, CL406, CL407, CL408, CL411, CL412, CL413, CL414, CL415, CL416, CL417, CL418, CL421, CL422, CL423

Instructions for use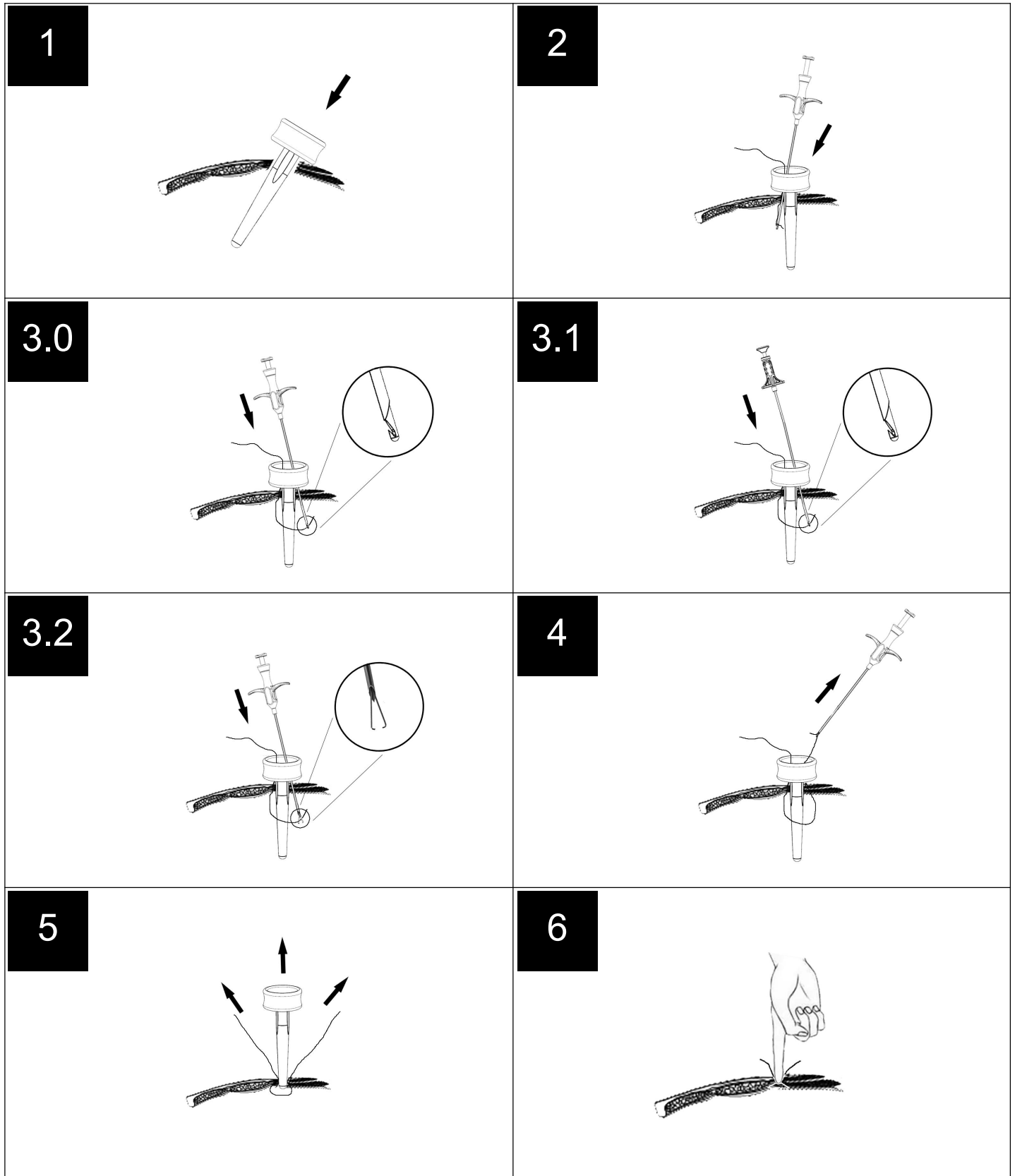

BEFORE USING THIS PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY.**IMPORTANT**

- This booklet is designed to assist using this product. It is not a reference to surgical techniques.
- This device was designed, tested and manufactured for single use patient use.
- Reuse of this device may lead to its failure and subsequent patient injury.
- Reprocessing and / or re-sterilization of this device may create risk of contamination and patient infection.
- Do not reuse, reprocess or re-sterilize this device.

INTENDED USE

The device is intended to approximate the edges of an internal wound or incision with sutures during laparoscopic surgery.

CONTRAINDICATIONS

- The use of Port Closure Device is contraindicated whenever endoscopic surgical techniques are contraindicated.
- The use of VHMED laparoscopic products is contraindicated whenever laparoscopic surgical techniques are contraindicated for any reason.

DESCRIPTION

- The instrument is composed of pushbutton, handle, needle, probe, hook and suture guides (if applicable).
- Sterile, Non-active, Transient.
- Shelf life: 3 years. The instrument has been sterilized by ETO

WARNINGS AND PRECAUTIONS

- This instrument is used in minimal invasive surgery, and conducting the laparoscopic operation along with the use of endoscope and the accessories.
- It should only be operated by individuals with adequate training and knowledge about the endoscopic techniques.
- Before the operation, verify the compatibility of instruments and accessories from different manufacturers.
- Do not use if the sterile package is damaged.
- The device shall be disposed as medical waste after using.
- The instrument is not intended for use when endoscopic techniques are contraindicated.
- This instrument is intended for single patient use only. **DO NOT RESTERILIZE.** Dispose the used products as standard hospital biohazard control practices.
- There is no contraindication directly linked to the instrument till now. Doctor should have the idea whether this instrument can be used on the patient according to the patients' particular situation. For further information about techniques, complications and hazards, please consult the latest medical literature.

POTENTIAL COMPLICATIONS FOR ALL VHMED PRODUCTS

Potential complications associated with the use of laparoscopic devices include but are not limited to organ or vessel damage / perforation, hemorrhage, hematoma, trocar site hernia, and septicemia / infection.

GENERAL INSTRUCTIONS FOR USE

1. Insert the suture guide with the holes aligned perpendicular to the trocar wound. **Refer to picture 1**
2. With a loaded suture passer, push the suture passer through fascia, muscle and peritoneum into the abdomen.

- Drop the suture and remove the suture passer. **Refer to picture 2**
3. On the opposite side of the guide, push the suture passer through all of the layers and pick up the suture with the hinged jaw or hook. **Refer to picture 3.0/3.1/3.2**
 4. Pull the suture up through the peritoneum, muscle, fascia and guide. **Refer to picture 4**
 5. Remove the guide and tie off the suture. Push down the knot to completely close the incision. **Refer to picture 5/6**

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VHMED (Nantong) Co. Ltd.
Very Heal Ltd.