
Laparoscopic Instrument


LS8001, LS8001S, LS8002, LS8002T

Instructions for use

Blunt Tip Dissector

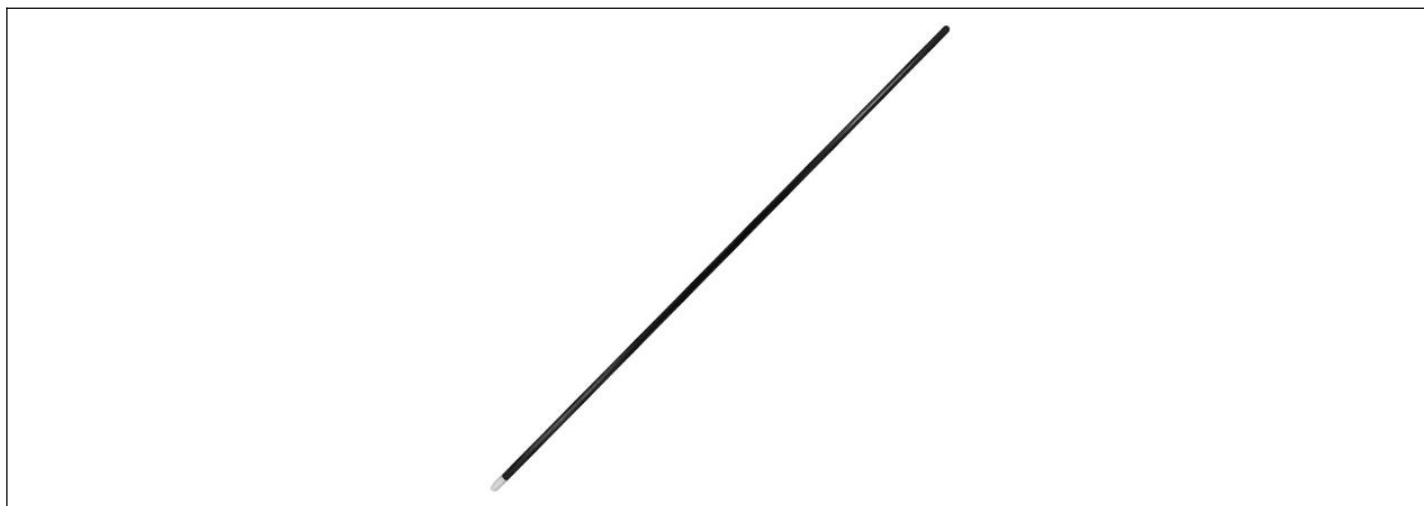Aspiration Needle

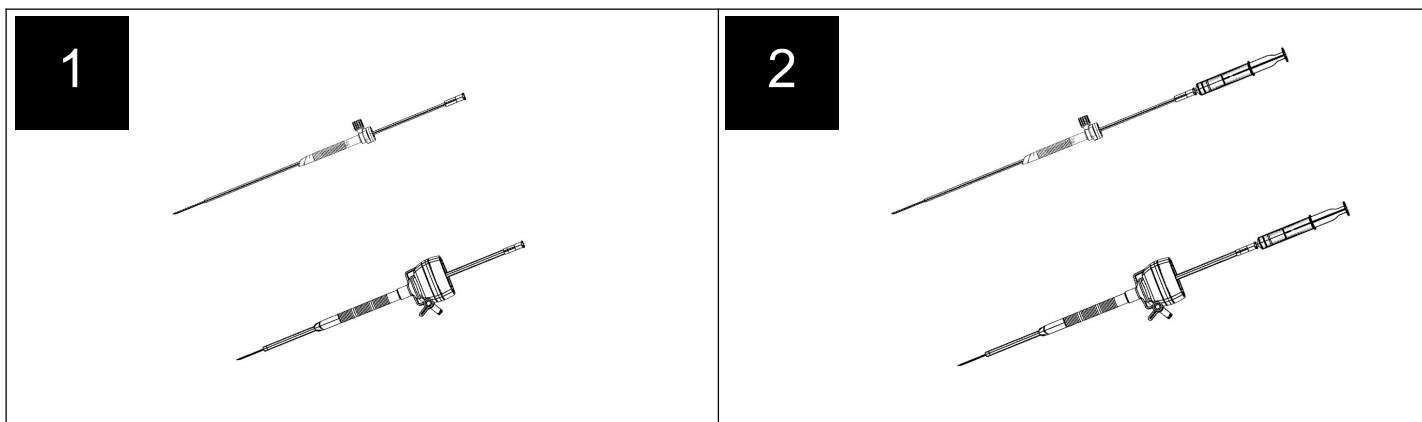**EN - ENGLISH****BEFORE USING THIS PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY.****IMPORTANT**

- This booklet is designed to assist using this product. It is not a reference of surgical techniques.
- This device was designed, tested and manufactured for single patient use.
- Reuse of this device may lead to its failure and subsequent patient injury.
- Reprocessing and / or re-sterilization of this device may create risk of contamination and patient infection.
- Do not reuse, reprocess or re-sterilize this device.

INTENDED USE

Blunt Tip Dissector

Single use blunt tip dissector is a sterile device intended to be inserted through a trocar for safe, atraumatic tissue manipulation and dissection during various surgical procedures.

Aspiration Needle

Single use aspiration needle is a sterile device intended to be inserted through a trocar for the simple, safe aspiration of fluids from cysts and organs inside of the patient cavity during various surgical procedures.

CONTRAINDICATIONS

The use of VHMED laparoscopic products is contraindicated whenever laparoscopic surgical techniques are contraindicated for any reason.

Use with caution if patients are allergic to raw materials contacting them directly, such as cotton or fiberglass.

DESCRIPTION**Blunt Tip Dissector**

- It consists of a small sponge made of soft and absorbent material (cotton) and a working shaft made of non-conductive material (fiberglass+FEP)
- Working parameter: $\phi 5 \times 450\text{mm}$;
- Shelf life: 3 years.

Aspiration Needle

- Working parameter: $\phi 3 \times 340\text{mm}$, $\phi 5 \times 340\text{mm}$;
- Shelf life: 3 years.

WARNINGS AND PRECAUTIONS

- Before the operation, verify the integrity of the instrument.
- Product should not be used if sterile packaging is damaged or opened. If damage is found, remove the product from the operative area to prevent unintended use, and return to manufacturer.
- This device is for single use only. It is not intended for disinfection and/or subsequent re-use, which can result in microbial contamination causing health deterioration of the patient.
- This device has not been evaluated for reprocessing or re-sterilization. Reprocessing and/or re-sterilization may damage the device, rendering it unusable and/ or may lead to device failure, which could result in patient illness, injury or death.
- This device shall be used in accordance with physician instructions, hospital protocols and this IFU. Read all sections of the IFU prior to use. Improper use of these devices may cause serious injury.
- This instrument is intended for single use only. DO NOT REUSE. Dispose the used products as standard hospital practices for biohazard control.

POTENTIAL COMPLICATIONS FOR ALL VHMED PRODUCTS

Potential complications associated with the use of laparoscopic devices include but are not limited to organ or vessel damage/perforation, hemorrhage, hematoma, trocar site hernia, and septicemia/infection.

GENERAL INSTRUCTIONS FOR USE (Blunt Tip Dissector)

1. Open the sterilized package, and introduce the device into patient cavity through appropriately sized trocar sleeve.

GENERAL INSTRUCTIONS FOR USE (Aspiration Needle)

1. Open the sterilized package, and introduce the device into patient cavity through appropriately sized trocar sleeve.

Refer to picture 1

2. Connect to a syringe and aspiration of fluids from cysts and organs. **Refer to picture 2**

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VHMED (Nantong) Co. Ltd.
Very Heal Ltd.