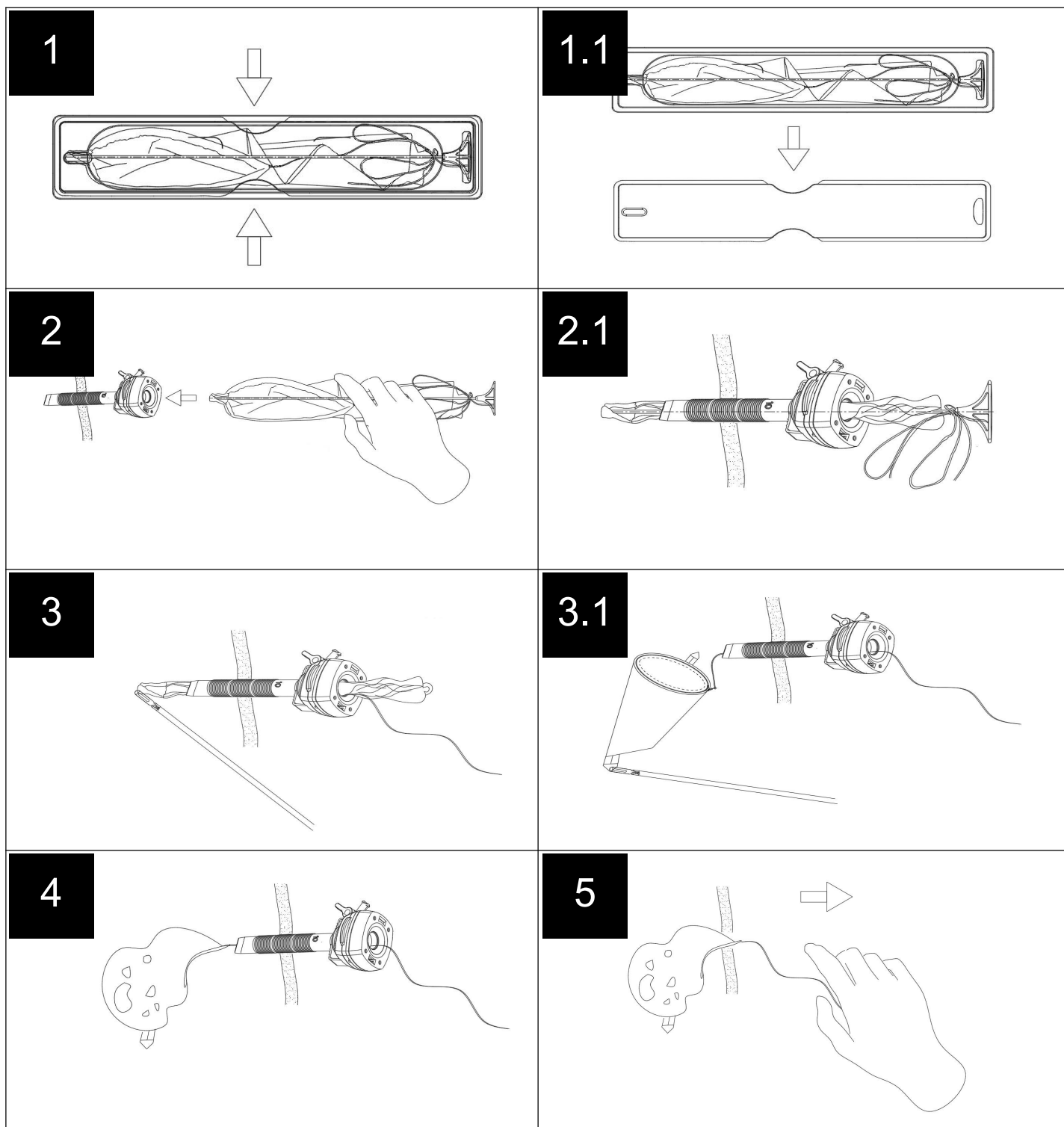
Disposable Specimen Retrieval Bag

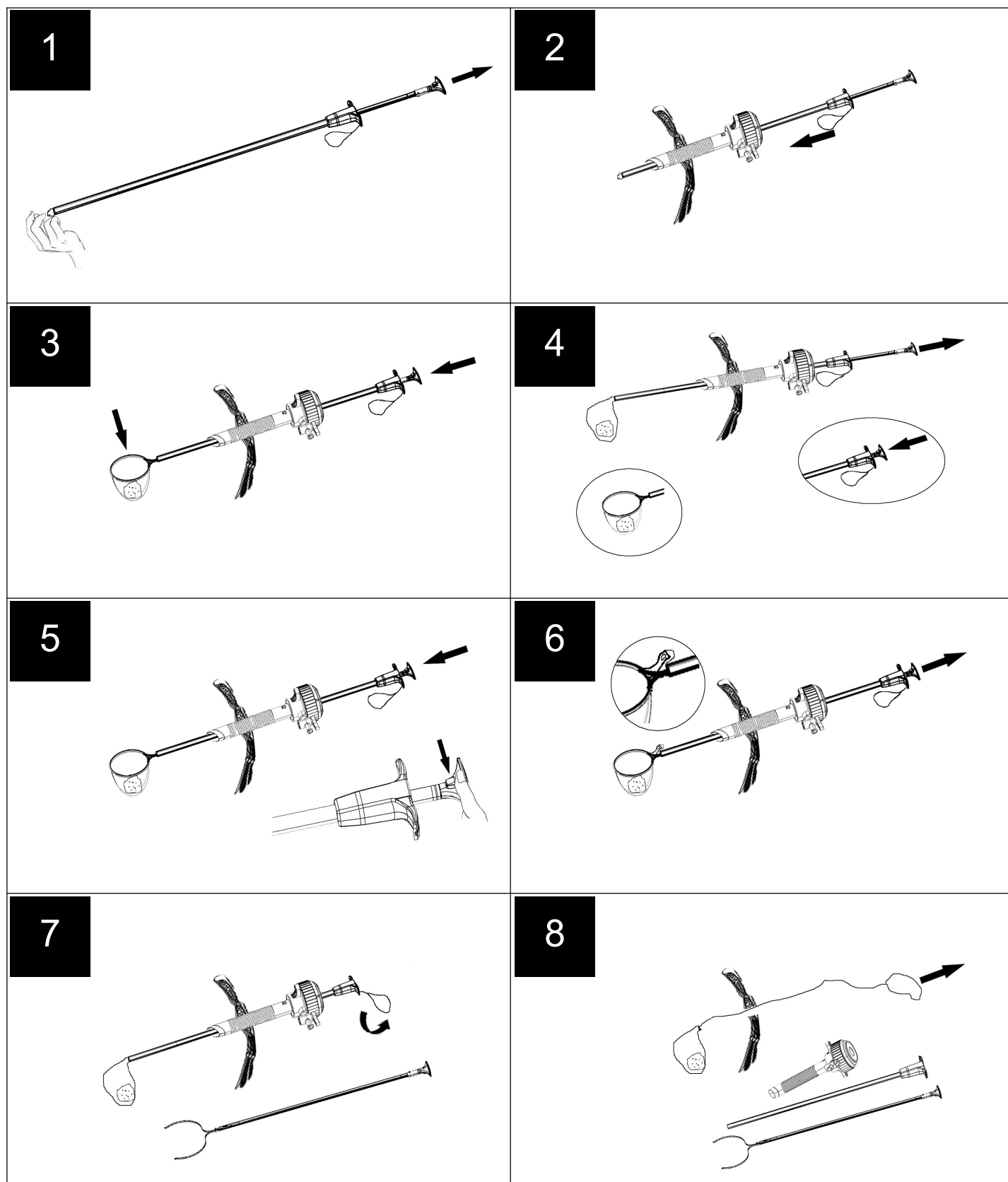
SR001, SR002, SR003, SR004, SR005, SR006, SR007, SR008, SR009, SR010, SR100, SR101, SR102, SR103, SR104, SR105, SR200, SR201, SR202, SR203, SR204, SR205

Instructions for use


ELABAG® (SR200, SR201, SR202, SR203, SR204, SR205)


FISHINGMAN[®] (SR100, SR101, SR102, SR103, SR104, SR105)

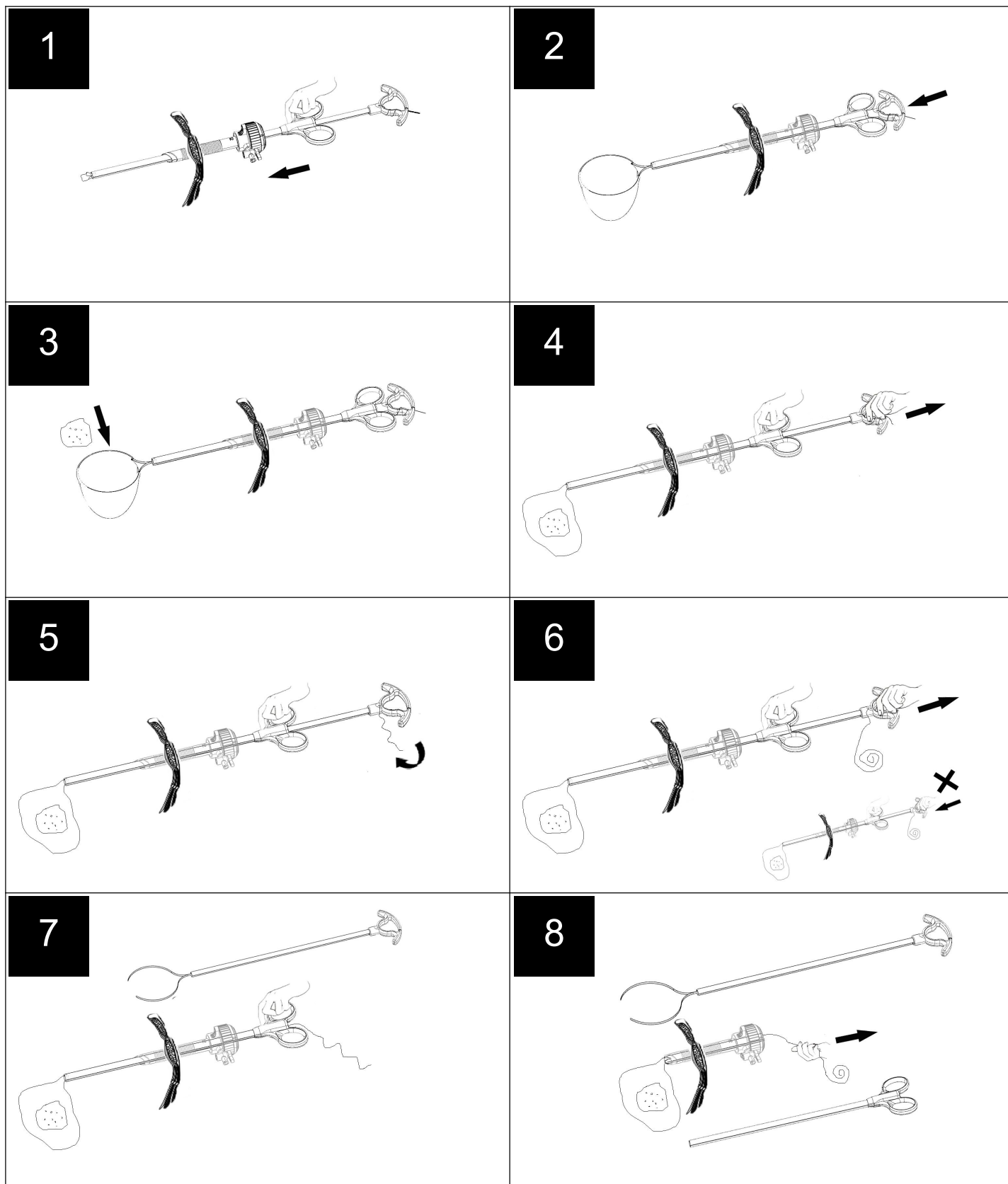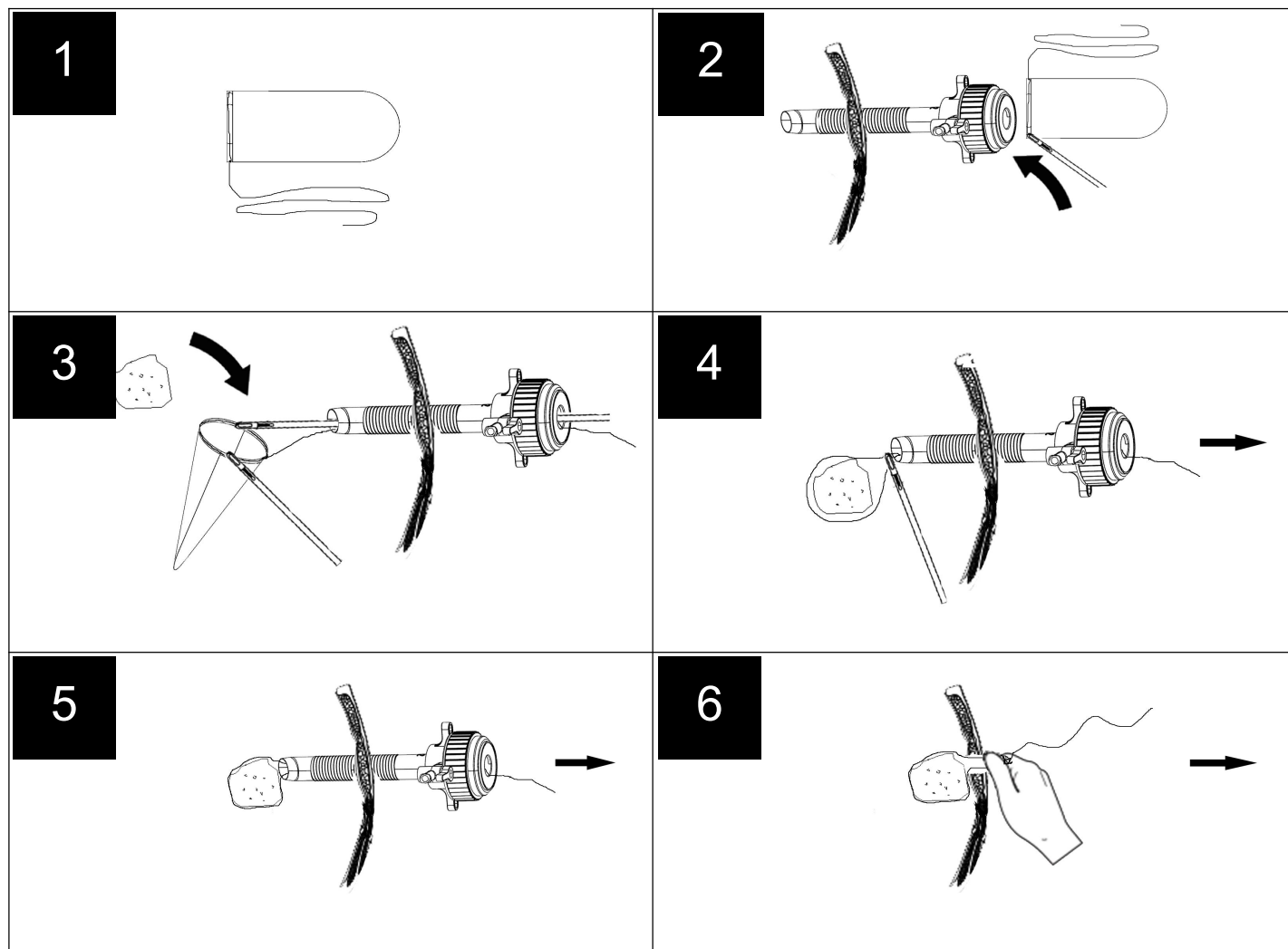Black & White + Endobags (SR001, SR002, SR003, SR004, SR005, SR006, SR007, SR008)

EN - ENGLISH

BEFORE USING THIS PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY.

IMPORTANT

- This booklet is designed to assist using this product. It is not a reference of surgical techniques.
- This device was designed, tested and manufactured for single patient use.
- Reuse of this device may lead to its failure and subsequent patient injury.
- Reprocessing and / or re-sterilization of this device may create risk of contamination and patient infection.
- Do not reuse, reprocess or re-sterilize this device

INTENDED USE

Disposable endobag is intended for use of retrieving of specimen from the cavities in minimal invasive surgery through trocar cannula. It largely increases the efficiency of removing the specimen or sample from the patient cavity.

CONTRAINDICATIONS

The use of VHMED laparoscopic products is contraindicated whenever laparoscopic surgical techniques are contraindicated for any reason.

DESCRIPTION

- This instrument is available with tether and tether-free configurations.
- Shelf life: 3 years. The instrument has been sterilized by ETO.

WARNINGS AND PRECAUTIONS

- Do not open the bag until it entirely enters into the patient cavity.
- When loading the tissues into the bag, prevent the bag from damaging by any sharp instruments.
- Do not drag the bag with sharpen instrument when removing the bag from cavity. Once the bag is damaged, the strength will be largely reduced.
- When removing the bag, it may balloon if the bag opening is tied tightly. Loosen the string slightly, and then use instrument squeeze to release the air. Do not break the bag in the process.
- The instrument is not intended for use when endoscopic techniques are contraindicated.
- This instrument is intended for single patient use only. DO NOT RESTERILIZE. Dispose the used products as standard hospital biohazard control practices.
- There is no contraindication directly linked with the instrument till now. Doctor should have the idea whether the instrument can be used on the patient according to the patients' particular situation. For further information about techniques, complications and hazards, please consult the latest medical literature.

POTENTIAL COMPLICATIONS FOR ALL VH MED PRODUCTS

Potential complications associated with the use of laparoscopic devices include but are not limited to organ or vessel damage / perforation, hemorrhage, hematoma, trocar site hernia, and septicemia / infection.

GENERAL INSTRUCTIONS FOR USE (ELABAG®)

1. Open the sterilized package with ad-hoc sterile procedure, and select an appropriately sized trocar sleeve..
Refer to picture 1/1.1
2. Introduce the device into the patient cavity through the trocar sleeve by the provided introducer. Ensure the bag reaches a suitable place. **Refer to picture 2/2.1**
3. Release the tether, remove the introducer and make the bag fully expand by graspers. **Refer to picture 3/3.1**
4. Put the specimen, diseased tissue, or foreign body into the bag with hand instruments. Once the specimen is completely placed into the bag, close the bag by drawing the closure tether. **Refer to picture 4**
5. Carefully remove the trocar cannula and the bag from the access site while maintaining the firm grip of the bag closure tether. **Refer to picture 5**

GENERAL INSTRUCTIONS FOR USE (FISHMANN®)

1. Check the integrity of the packaging and open the sterilized package, and then pull the rear handle till the entire bag retracted inside the shaft, use hand assist if necessary. **Refer to picture 1**
2. Introduce the device into patient cavity through appropriately sized trocar sleeve. **Refer to picture 2**
3. Ensure the bag reach the desired location, keep the front handle stable, and push forward the rear handle till the bag fully expands in the patient cavity. **Refer to picture 3**
4. Put the specimen, diseased tissue or foreign body into the bag with hand instrument. Keep the front handle still, pull back the rear handle to close the bag. Repeat the procedures in step 3 & 4, specimen retrieval can be conducted multiple times with the same patient. **Refer to picture 4**
5. Keep the front handle stable, and push forward the rear handle while pressing the bag release button. **Refer to picture 5.** The bag retention mechanism will be deactivated and the bag will fall off from the instrument shaft. **Refer to picture 6**

6. Keep the front handle still, pull the rear handle out of the instrument. **Refer to picture 7**
7. Remove the front handle out of the trocar cannula. Finally remove the bag together with trocar sleeve, or remove the trocar sleeve first then remove the specimen from the incision, dilate the incision if necessary. **Refer to picture 8**

GENERAL INSTRUCTIONS FOR USE (Black & White + Endobags)

1. Open the sterilized package, and introduce the device into patient cavity through appropriately sized trocar sleeve. **Refer to picture 1**
2. Ensure the bag has reached the suitable place, keep the front handle stable, and push forward the rear handle till the bag fully expands. **Refer to picture 2**
3. Put the specimen, diseased tissue or foreign body into the bag with a hand instrument. **Refer to picture 3**
4. For Non-Detachable Endobag, keep the front handle stay still, pull back the rear handle to close the bag. And remove the bag together with the trocar sleeve. For Detachable Endobag, keep the front handle stay still, pull back the rear handle to close the bag. **Refer to picture 4**
5. Loosing the drawing tether. **Refer to picture 5**
6. Keep the front handle stay still, pull the rear handle out, and close the bag by drawing the tether. **Refer to picture 6**
7. Remove the front handle. **Refer to picture 7**
8. Finally remove the bag together with the trocar sleeve, or remove the trocar sleeve first then remove the specimen from the incision, dilate the incision if necessary. **Refer to picture 8**

GENERAL INSTRUCTIONS FOR USE (VSIMPLY – Bag Only)

1. Open the sterilized package with ad-hoc sterile procedure, and select appropriately sized trocar sleeve. **Refer to picture 1/1.1**
2. Introduce the device into the patient cavity through appropriately sized trocar sleeve. Ensure the bag reaches the suitable place to allow it fully expands. **Refer to picture 2/2.1**
3. Then put the specimen, diseased tissue or foreign body into the bag with hand instrument. **Refer to picture 3/3.1**
4. Once the specimen is completely placed into the bag, close the bag by drawing the closure tether. **Refer to picture 4/4.1**
5. When the specimen is completely within the bag and extraction is desired, pull the bag closure tether till the leading edge of the bag is inside the tip of the trocar cannula and bag closure tether is outside of the trocar cap. **Refer to picture 5/5.1**
6. Carefully remove the bag and trocar cannula from the access site while maintaining the firm grip of the bag closure tether. **Refer to picture 6/6.1**

NOTE: Do not attempt to pull the specimen through the trocar cannula.

NOTE: If the bag and its contents are too large to be extracted, carefully dilate the incision to facilitate easy bag removal.

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VH MED (Nantong) Co. Ltd.
Very Heal Ltd.