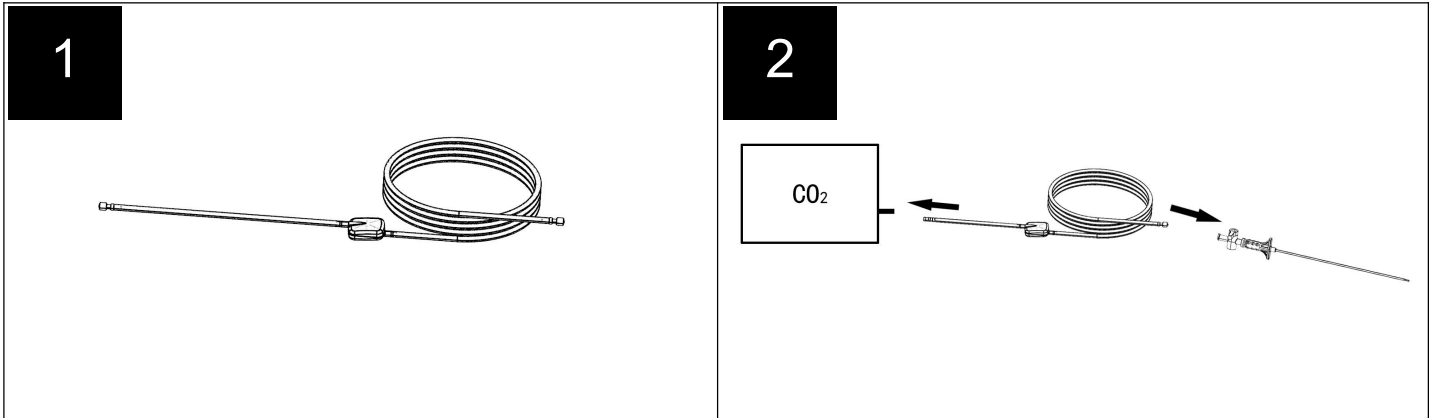
Disposable Insufflation Tube

TB001, TB002, TB003, TB004, TB005, TB006, TB007, TB008, TB009

Instructions for use

**EN - ENGLISH**

BEFORE USING THIS PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY.

IMPORTANT

- This booklet is designed to assist using this product. It is not a reference of surgical techniques.
- This device was designed, tested and manufactured for single patient use.
- Reuse of this device may lead to its failure and subsequent patient injury.
- Reprocessing and / or re-sterilization of this device may create risk of contamination and patient infection.
- Do not reuse, reprocess or re-sterilize this device

INTENDED USE

The device is intended to be used to maintain pneumoperitoneum and filter CO₂ during laparoscopic surgery.

CONTRAINDICATIONS

- Patients with cardiovascular disease who cannot bear the artificial pneumoperitoneum.
- Patients who cannot bear laparotomy.
- Patients with coagulation disorders.
- Patients with severe liver cirrhosis and portal hypertension.
- The use of VHMED laparoscopic products is contraindicated whenever laparoscopic surgical techniques are contraindicated for any reason.

DESCRIPTION

- The instrument is composed of joint, tube and filter.
- Sterile, Non-active, Transient.
- Shelf life: 3 years. The instrument has been sterilized by ETO.

WARNINGS AND PRECAUTIONS

- This instrument is used in minimal invasive surgery, and conducting the laparoscopic operation cooperated with endoscope and the accessories.

- It should only be operated by individuals with adequate training and knowledge about the endoscopic techniques.
- Before the operation, verify the compatibility of instruments and accessories from different manufacturers.
- Do not use if the sterile package is damaged.
- The device shall be disposed as medical waste after using.
- The instrument is not intended for use when endoscopic techniques are contraindicated.
- This instrument is intended for single patient use only. **DO NOT RESTERILIZE**. Dispose the used products as standard hospital practices for biohazard control.
- There is no contraindication directly linked with the instrument till now. Doctor should have the idea whether this instrument can be used on the patient according to the patients' particular situation. For further information about techniques, complications and hazards, please consult the latest medical literature.

POTENTIAL COMPLICATIONS FOR ALL VHMED PRODUCTS

Potential complications associated with the use of laparoscopic devices include but are not limited to organ or vessel damage/perforation, hemorrhage, hematoma, trocar site hernia, and septicemia/infection.

GENERAL INSTRUCTIONS FOR USE

1. Open the sterile package and select appropriately sized trocar and insufflator. **Refer to picture 1**
2. Connect the joint 1 (long end for tube with filter) to a suitable trocar and the joint 2 (short end for tube with filter) to a suitable insufflator. Start the insufflation with a comfortable gas pressure value after checking the system has been connected properly. A pressure-limited gas [typically CO₂] will be supplied from the insufflator to the surgical cavity to establish a pneumoperitoneum. When the gas flows through the filter, potential impurity can be filtered. **Refer to picture 2**

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VHMED (Nantong) Co. Ltd.
Very Heal Ltd.